

BATTERY GUIDE FREESTYLE® COMFORT®

Getting Started

1) Install the Battery Pack:

A. Align the battery with the bottom housing from the center of the FreeStyle Comfort. **B.** Slide the battery inward until you hear a click to confirm that it has locked in place.



2) Connect External Power



Connect one end of the AC power cord to the inlet on the power supply brick. Connect the other end to a wall outlet.



Shown Above is the FreeStyle Comfort Power Supply



Charging the Battery

The FreeStyle Comfort will recharge the battery pack anytime it is installed in the concentrator and the concentrator is connected to an external AC or DC power source, unless placed in Airplane Mode.

Battery charge will be displayed on the display screen. If off, press up or down to display battery percentage.

100% is fully charged.

To check the battery charge when it is not installed in the FreeStyle Comfort, lay it flat and press the TEST button. The battery gauge indicator lights (25-100%) will illuminate to the right of the TEST button to indicate the level of the battery pack charge.



Battery Recharge Time:
Single Battery: 3.5 Hours
Double Battery: 6.0 Hours



Call Us Toll-Free Today!

1-877-774-9271

oxygenconcentratorstore.com