

Important Patient Information

NEW!

LifeChoice
ACTIVOX™
breaking boundaries

**“I have
the
power.”**



AMERICAN
MEDICAL

AMSR

SALES AND
RENTALS



1-877-774-9271 | www.oxygenconcentratorstore.com

You have the power.

Stay Active

The LifeChoice Activox POC is the ideal oxygen therapy for the active patient. The system is lightweight and provides the longest internal battery available in a POC, empowering true freedom and mobility.

- Lightweight at only 4.8 lbs.*
- No carts or tanks required
- Up to 12 hours of portability with Pro Model
- Provides up to 3 LPMeq
- FAA-approved
- Optional 3-hour external battery available

*Pro Model Weight



Sleep Well

Unique Auto Mode Technology automatically adjusts to changes in breathing patterns at rest to provide the power of oxygen on-demand throughout the night, anytime, anywhere.

ACTIVOX[™]
LifeChoice
breaking boundaries

Picture living your life to its fullest.



Power to Stay Active

At only 4.8 lbs., the LifeChoice Activox POC offers total hands-free mobility. The system does not require a cart or a tank and provides the longest internal battery available in a POC, up to 12 hours with the Pro Model.* The system features unique Auto Mode and PULSE-WAVE™ Technologies which work together to deliver oxygen around the clock, keeping you saturated day or night.

The LifeChoice Activox POC is the perfect travel companion: it is FAA-approved for domestic and international airline travel and operates up to 10,000 ft. It is powered and recharged by AC and DC adapters for in-home and mobile use and is capable of being recharged while functioning at any setting. With a LifeChoice Activox POC, no carts, refills or monthly service calls from a provider are required!

*See specifications chart for full battery details.

Power That Delivers

The LifeChoice Activox POC features Auto Mode Technology which automatically adjusts patient therapy based on oxygen demand.

- During Activity – Auto Mode supports higher respiration rates
- During Sleep – Auto Mode adjusts to be more sensitive in detecting breaths to maintain saturation throughout the night

Power of the PULSE-WAVE™

The LifeChoice Activox POC utilizes unique PULSE-WAVE™ Technology that delivers oxygen continuously during the first 30% of the inspiratory period so that all of the oxygen delivered is actually received. This delivery also helps minimize uncomfortable side effects.

Power That's Affordable

The LifeChoice Activox POC is reimbursed by Medicare and most private insurance plans. The Standard Package includes all the necessary elements for easy use and portability:

- LifeChoice Activox POC
- 12-Hour Internal Battery at 1 LPMeq with Pro Model; 5.5-Hour Internal Battery at 1 LPMeq with Sport Model
- Versatile 4-Way Carrying Case (Backpack, Shoulder Strap, Waist Pack, Briefcase)
- AC/DC Adapter/Charger
- 7-Foot Nasal Cannula
- Accessory Bag
- Standard 3-Year Limited Warranty (1 Year for Battery, Sieve Bed and Accessories)

Optional 3-hour external battery available for purchase.



NEW!

LifeChoice
ACTIVOX
breaking boundaries



LifeChoice
ACTIVOX
breaking boundaries

1-877-774-9271 | www.oxygenconcentratorstore.com

American Medical Sales and Repair
7042 S Revere Parkway
Suite 450
Centennial, CO 80112

Weight:	Sport Model: 4.6 lbs. Pro Model: 4.8 lbs.		
Dimensions:	9.05" w x 7.875" h x 4.38" d		
Mode Of Operation:	Stationary/Portable		
Flow Control Pulse Settings:	1 LPMeq	2 LPMeq	3 LPMeq
Average Sound Level at 10 ft. (3 m):	41 dB	44 dB	46 dB
Nominal Wave Volumes:	10 ml	20 ml	30 ml
Approx. Internal Battery Duration: *	Sport Model: 5.5 Hours Pro Model: 12 Hours	4 Hours 6 Hours	2 Hours 5 Hours
Approx. External Battery Duration: *†	3 Hours	3 Hours	2 Hours
Power Supply:	AC 100-240 VAC 50-60 Hz Cycles DC 12-19 VDC Battery 12-16.8 VDC		
Approx. Internal Battery Recharge Time:	Sport Model: 3 Hours Pro Model: 4 Hours		
Approx. External Battery Recharge Time:	3 Hours		
Warm-up Time:	30 Seconds		
Oxygen Concentration:	90% (+/- 3%)		
Oxygen Concentration Sensor Alarm:	85% or Less		
Operational Humidity Range:	15% to 95%, Non-condensing		
Operational Temp. Range:	41°F to 104°F (5°C to 40°C)		
Operational Altitude Max:	10,000 ft. (3,048 m)‡		
Storage Humidity:	Up to 95%, Non-condensing		
Storage Temp. Range:	32°F to 140°F (0°C to 60°C)		
LifeChoice Activox Warranty:	3 Years (1 Year for Batteries, Sieve Bed and Accessories)		



Now Available At

*Based on Breaths Per Minute.

†Not included in Standard Package.

‡Performance degradation may occur at high altitude.

American Medical Sales and Rentals
7032 S Revere Parkway
Suite 320
Centennial, CO 80112

1-877-774-9271 | www.oxygenconcentratorstore.com