



Cartridge Replacement

# STEP 1

## Removing Cartridge.



1 Turn off the X-PLO<sub>2</sub>R™ by pressing the power button for 2 seconds.



2 Remove the X-PLO<sub>2</sub>R from the carrying bag, if applicable.



3 Remove the battery from the X-PLO<sub>2</sub>R.



4 Place the X-PLO<sub>2</sub>R on its side to access the underside of the POC.



5 Pull the cartridge out of the POC by pulling on the cartridge pull handle.



6 Remove the cartridge completely from the X-PLO<sub>2</sub>R and throw away.

↪ Turn sheet over to continue to step 2.

By: **belluscura**<sup>®</sup>  
LBL 5026-02

1-877-774-9271 | [www.oxygenconcentratorstore.com](http://www.oxygenconcentratorstore.com)



Cartridge Replacement

# STEP 2

## Installing Cartridge.



1 Remove cartridge from mylar bag. The new cartridge has a dust cap on top. Remove the dust cap. Make sure there is no dust or debris where the dust cap was located.



2 Insert new cartridge top first into the X-PLO<sub>2</sub>R. Do not leave the cartridge ends exposed; the cartridge should be inserted into the X-PLO<sub>2</sub>R as soon as the used cartridge is removed.



3 Push the cartridge into the X-PLO<sub>2</sub>R so it is fully inside.



4 The cartridge pull handle should lay flat once the cartridge is secured.



5 Connect charged battery to the X-PLO<sub>2</sub>R.



6 Turn on the X-PLO<sub>2</sub>R by pressing the power button for 2 seconds. Set the flow setting to your prescribed setting and use normally.

### American Medical Sales and Rentals

15055 E. Hinsdale Drive, Bldg D  
Centennial, CO 80112  
Phone: +1 (877) 774-9271



By: **belluscuro**<sup>®</sup>  
LBL 5026-02

[www.oxygenconcentratorstore.com](http://www.oxygenconcentratorstore.com)  
E-mail: [sales@amsrco.com](mailto:sales@amsrco.com)