

# CHECK LIST

Locate the following items in your X-PLO<sub>2</sub>R<sup>®</sup> box to get started:



X-PLO<sub>2</sub>R Unit Includes:

- Air intake filter
- Battery



Carrying Bag  
(The unit arrives already placed inside the bag)



Cartridge



User Manual



AC Power Supply



DC Power Supply



Handle  
(Attaches to Carrying Bag)



Shoulder Strap  
(Attaches to Carrying Bag)



4-Cell Battery or 8-Cell Battery



## TRAVEL WITH X-PLO<sub>2</sub>R

The X-PLO<sub>2</sub>R lets you travel with ease. It's perfect for carrying on buses, trains, cruise ships and is even FAA compliant for air travel. And with supplemental oxygen and direct-charge batteries, exploring has never been so easy.

1-877-774-9271

[www.oxygenconcentratorstore.com](http://www.oxygenconcentratorstore.com)

©2021 Belluscura. All Rights Reserved.



## Quick START GUIDE

By: **belluscura**<sup>®</sup>



# Quick START GUIDE

X-PLOR<sup>®</sup>

By: belluscura<sup>®</sup>

## DO NOT TURN ON THE X-PLOR<sub>2</sub>R UNTIL YOU REACH STEP 4.

It is quick and easy to get your X-PLOR<sub>2</sub>R started, but please read the entire User Manual before using.



### LOCATE THE BATTERY

- Your X-PLOR<sub>2</sub>R unit will arrive inside the carrying bag, which zips open at the bottom for easy access to the battery.
- To disconnect the battery, simply push the release latch down and slide the battery toward you.



### REMOVE DUST CAP & INSERT CARTRIDGE

- Remove the dust cap from the cartridge.
- Then insert the cartridge all the way into the unit so the handle lays flat.
- To reattach the battery, slide it back on until the latch returns to locked position and zip the bag.



### CHARGE THE BATTERY

- Place in a well-ventilated area to fully charge the battery before using your X-PLOR<sub>2</sub>R, which can take up to 5 hours for the 4-cell battery and up to 6 hours for the 8-cell battery.
- To charge, insert the AC power connector into the battery's power receptacle and the other end into an electrical outlet.



### TURN ON & SET OXYGEN FLOW

- When powered on, position the unit upright to warm-up (warm-up time is less than 2 minutes.)
- Press the POWER button for 2 seconds to turn on your X-PLOR<sub>2</sub>R.
- Use the UP/DOWN ARROWS to set the oxygen flow level, as specified in your oxygen prescription.



### NASAL CANNULA

- NOTE: Cannula must be purchased separately.
- Connect the nasal cannula to the oxygen output nozzle located on top of the X-PLOR<sub>2</sub>R.
- Place the other end in your nostrils. Oxygen will be delivered through the nasal cannula with each breath.
- Remember to breathe through your nose as normal.



### GO X-PLOR!

- Attach the shoulder strap and handle to your X-PLOR<sub>2</sub>R device and enjoy your new found mobility!

**LIVE LIFE. BREATHE EASY.<sup>™</sup>**  
**GO X-PLOR<sub>2</sub>R.**