

STOP!

www.oxygenconcentratorstore.com

INOGEN ONE BATTERY CARE AND MAINTENANCE INSTRUCTIONS.

Please follow these important guidelines to maximize battery life and performance:

- 1. Always keep liquids away from batteries. If batteries become wet, discontinue use immediately and dispose of battery properly.**
- 2. If using multiple batteries, make sure that each battery is labeled (1, 2, 3, or A, B, C, etc) and rotated on a regular basis. Batteries should not be left dormant for extended periods of time.**
- 3. For long term storage, keep battery in a cool, dry place. Store with a charge of 40-50%**
- 4. Battery model #BA-100 only, for use with model #IO-100**
 - a. Remove battery from the Inogen One while not in use.**
 - b. Once pr month: Fully charge the battery then discharge to 10% and charge back to full.**

Note: Non-compliance with these guidelines will reduce battery life and may void your warranty.

www.oxygenconcentratorstore.com