

# 6 Quick Steps to Recharge Your Battery



## INOGENONE<sup>®</sup>G5 External Battery Charger Guide

American Medical Sales and Rentals  
7032 S Revere Parkway, Suite 320  
Centennial, CO 80112  
Phone: 877-774-9271

[oxygenconcentratorstore.com](http://oxygenconcentratorstore.com)  
E-mail: [sales@amsrco.com](mailto:sales@amsrco.com)



# INOGEN ONE<sup>®</sup> G5 External Battery Charger

Your Inogen One<sup>®</sup> G5 External Battery Charger (BA-503) will charge your Inogen One<sup>®</sup> G5 single and double batteries. It takes approximately 4.5 hours to charge a fully depleted single battery, and approximately 9 hours to charge a fully depleted double battery. Note: the battery will also recharge in the Inogen One<sup>®</sup> G5 when powered by the AC Power Supply, or DC Power Cable.



1 Plug the External Battery Charger AC power supply cord into an electrical outlet.



2 Plug the External Battery Charger AC power supply into the battery charger.




3 Slide the charger onto the Inogen One<sup>®</sup> G5 Battery by clicking and locking into the charger.



4 When the battery is in the correct position, a solid red light will indicate that the battery is charging.



5 When the green light illuminates, the battery is fully charged\*.

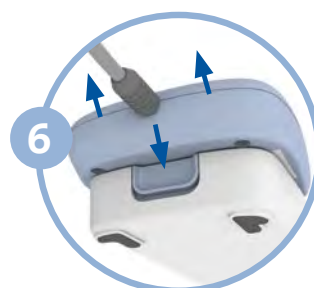

6 Press battery latch down and slide charger off battery.

\*If the red light is flashing, unplug the unit and complete steps 2-3 again. If a red flashing light continues after these steps, contact your Equipment Provider.